

# Acufast Needle Tray Kit

## Included in your kit:

- 1 metal tray
- 4 metal strips pre attached
- 4 Acuclips
- 4 Acubases
- 4 additional metal strips in brown envelope (use these for additional parking spots for your Acubases in your clinic room)

## Instructions:

1. Remove the blue protective plastic layer on the metal strips pre-arranged on the tray.
2. Remove the Acubases from their boxes and place them the strips
3. Place the Acuclip on the Acubases.
4. Each of the Acubase and Acuclip combo will hold one pouch of your most commonly sized needles. In the picture, we have 15mm, 30mm, 40mm and 50mm as our high turnover needles.
5. The top row of the tray have the strips closest to the top edge of the tray. These are for your shorter needles
6. The second row has the strips below the top and are for your longer needles.
7. For the 15 and 30 mm needles we suggest you label your clips with a marker and cover with a small piece of tape to prevent the ink from being rubbed off. This way you can easily identify the thickness and length of the needles. The Acuclips for the longer needles do not need to be identified like the shorter ones as the sizing is visible on the lower end of the pouch. (see picture)
8. At the beginning of your clinic day prepare your tray with the pouches and tubes (if you use them). Then cycle through your pouches as you proceed through your day. An open pouch, clipped as shown below maintains a clean field for 12 hrs.

